

# NORDIC CAB



## Split seat

## Assembly instructions




1. Remove pillows and safety harnesses from left side of original ones (front view)
2. Cushions and harness are attached to the new seat part in the same way

3. Thread the lower long strap through the opening at the bottom of the original seat
4. Thread the strap under all openings and around to the opposite set. This is because it should not get uncomfortable to sit.



5. Take the strap around the entire seat and attach the buckle to the back of the seat.
6. Tighten

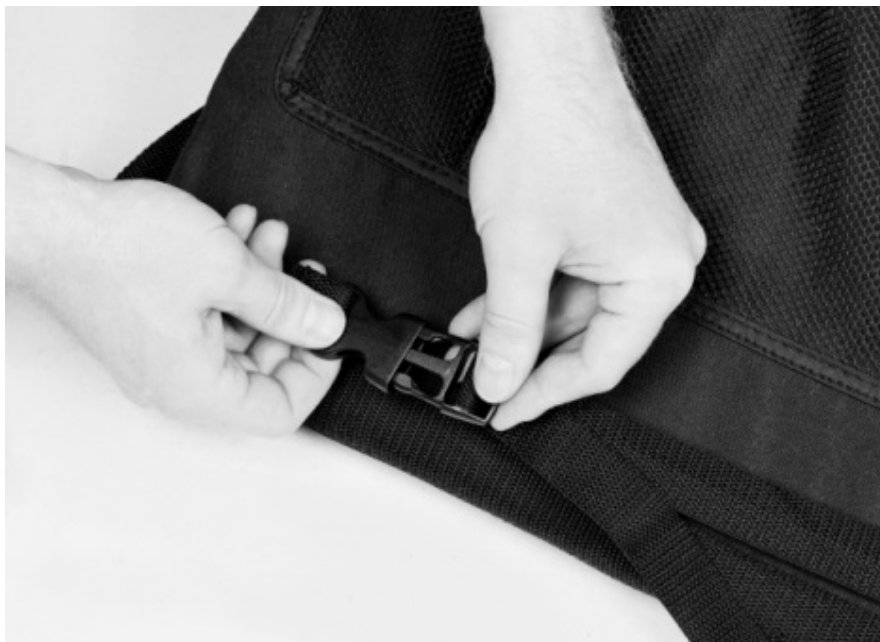

7. Take the strap at the top of the split seat over the upper cross bar, and attach the buckles
8. Tighten



***IMPORTANT!***

**DO NOT use in Nordic Cab as a bicycle trailer**

**Mounted on the left side of the carriage (seen from the front)**

***REMEMBER!***

Make sure braces, straps and buckles are attached and tightened.  
Always wear safety belts.